

Zuckerberg San Francisco General Hospital

Point of Care Testing

Urine Pregnancy QC & Patient Test Record

Unit _____

Recording of Patient Results:

1. Verify the Lot # and Expiration Date.
2. Cloudy urine should be allowed to settle. Use clear urine for testing.
3. Refrigerated urine must be equilibrated to 15 – 30 degrees Celsius before testing.
4. Test device should remain in the sealed pouch until ready for use.
5. Patient Results: Negative: a line appears ONLY in the C (control) window.
Positive: a line appears in the T (test) and C (control) windows. ANY line in the T window should be considered Positive. Test lines may be lighter or darker than in the C window.
6. Record all patient and control window results. Repeat patient sample if invalid QC window observed.

READ BETWEEN 3-4 MINUTES

No.	Date:	UPREG Kit Lot # & Expiration Date	Patient MRN & ----- Name:	Control Window		PATIENT RESULTS (POS or NEG)	Test Performed by:
				Does a line appear in the control window? (Y/N)	Is the background clear: white to light pink? (Y/N)		
01.			Patient MRN ----- NAME				
02.			Patient MRN ----- NAME				
03.			Patient MRN ----- NAME				
04.			Patient MRN ----- NAME				
05.			Patient MRN ----- NAME				
06.			Patient MRN ----- NAME				
07.			Patient MRN ----- NAME				
08.			Patient MRN ----- NAME				
09.			Patient MRN ----- NAME				
10.			Patient MRN ----- NAME				
11.			Patient MRN ----- NAME				
12.			Patient MRN ----- NAME				
13.			Patient MRN ----- NAME				
14.			Patient MRN ----- NAME				

Reviewed By: _____