

**What is IQCP?**

“IQCP” stands for “**Individualized Quality Control Plan.**” As of Jan 1, 2016, CMS said **for non-waived tests, either run your external QC daily or implement IQCP.** The non-waived POCT tests affected are GEM4000 tests, ROTEM delta tests, GEM OPL, and ACT. We chose to do IQCP on our non-waived POCT tests.

**Why is the advantage of an IQCP over daily QC?** QC is run much less frequently (less time and less cost), but actions are taken to mitigate identified risks to ensure patient safety.

**What are the 3 key elements of the IQCP?**

1. Risk Assessment	2. Quality Control Plan	3. Quality Assessment
<p>Identify <b>what can go wrong, how bad and how often</b>, in all three phases (pre-analytical, analytical, and post-analytical) of the testing process.</p> <p><b>Think “STREP”:</b></p> <p><b>S</b> – Specimen: correct collection tube, collecting collection site, adequate mixing of the specimen</p> <p><b>T</b> – Test: follow the procedure, running CVP1 instead of CVP2 or High LQC instead of Low LQC, QC failures, errors, instrument failures</p> <p><b>R</b> – Reagents: degrading reagents, preparing of reagents</p> <p><b>E</b> – Environment: room temp, fridge temp, humidity</p> <p><b>P</b> – Personnel: training and competency, user verifies that a test order in place prior to testing</p> <p><b>Now, is there a need for action</b> (mitigation steps) to reduce the risk for acceptable patient safety?</p>	<p>Refers to <b>external QC testing</b> on the instrument.</p> <p>For example, we run CVP1, CVP2, CVP3 and CVP4 with every cartridge change on the GEM4000 and High and Low LQC on the Hemochron Signature Elite.</p>	<p>Monitors the ongoing effectiveness based on the risks identified. Review and evaluate at least monthly:</p> <ul style="list-style-type: none"> <li>• Errors</li> <li>• QC failures</li> <li>• Unusual occurrences</li> <li>• corrective actions</li> <li>• complaints from providers</li> </ul> <p>instrument and equipment maintenance and function checks</p> <p><b>Are there new risks?</b>  <b>Did those mitigation steps work for those risks identified last time?</b>  <b>Go back to #1 Risk Assessment and revise IQCP as needed.</b></p>

***If an inspector asks about the IQCP risks and quality assessment tool for your area, refer him/her to your department super-user and/or Point of Care team.***


**What part do you play in the QC Plan? I make sure that all quality controls pass prior to patient testing.**


	<u>GEM4000</u>	<u>HEMOCHRON SIGNATURE ELITE</u>	<u>GEM OPL</u>	<u>ROTEM</u>
I do...	<ul style="list-style-type: none"> <li>• Daily temperature checks (room and fridge)</li> <li>• With every cartridge change: Run external liquid controls (CVP1, CVP2, etc)</li> <li>• Annually: Competency assessments</li> <li>• Monthly: Method comparison (“PCOM”) samples are run to compare to the lab’s results.</li> <li>• Problems: Call Tech Support and notify the Super User/POCT.</li> <li>• NOTE: The system runs internal controls daily in the background.</li> </ul>	<ul style="list-style-type: none"> <li>• Daily: Check Electronic QC (EQC) and temperature (fridge and room)</li> <li>• Monthly and new lot of cuvettes: external liquid QC (LQC; normal and abnormal)</li> <li>• Annually: Competency assessments and Proficiency Testing</li> <li>• Problems: Call Tech Support and notify the Super User/POCT.</li> <li>• NOTE: EQC is run every 8 hours along with analyzer temp checks</li> </ul>	<ul style="list-style-type: none"> <li>• Daily: Optical QC filter tests (yellow and orange filters)</li> <li>• Weekly and new lots of cuvettes: external liquid controls (LQC; Multi-4 co-ox controls levels 1, 2 and 3).</li> <li>• Annually: Competency assessments</li> <li>• Problems: Call Tech Support and notify the Super User/POCT.</li> </ul>	<ul style="list-style-type: none"> <li>• Daily: Internal QC evaluated, preventive maintenance, and temperature checks (fridge and room)</li> <li>• Weekly and new lots: external liquid controls (LQC; RoTrol N and RoTrol P) and weekly preventive maintenance</li> <li>• Quarterly: preventive maintenance</li> <li>• Semi-annually: Proficiency Testing and preventive maintenance (also every 250 measurements)</li> <li>• Annually: Competency assessments and preventive maintenance</li> <li>• Triennially: preventive maintenance</li> <li>• Problems: Call Tech Support and notify the Super User/POCT.</li> </ul>
POCT does...	<ul style="list-style-type: none"> <li>• Method comparison and linearity studies semi-annually</li> <li>• Monitor temperatures (fridge and room) and humidity</li> <li>• Monthly review of QA and maintenance</li> </ul>	<ul style="list-style-type: none"> <li>• Method comparison study semi-annually</li> <li>• Monitor temperatures (fridge and room)</li> <li>• Monthly review of monthly attestations and maintenance</li> <li>• Departmental Super Users: Submit monthly attestations</li> </ul>	<ul style="list-style-type: none"> <li>• Method comparison and linearity studies semi-annually</li> <li>• Monitor temperatures (fridge and room)</li> <li>• Departmental Super User: Submits monthly attestations</li> </ul>	<ul style="list-style-type: none"> <li>• Monitor temperatures (fridge and room) and humidity</li> <li>• Departmental Super User: Submits monthly attestations</li> </ul>

### 3 ELEMENTS OF AN IQCP:



***RISK ASSESSMENT: What can go wrong? How badly (severity)? How often (frequency)? Is the risk acceptable for patient safety? If not, what actions (mitigation steps) need to be taken?***

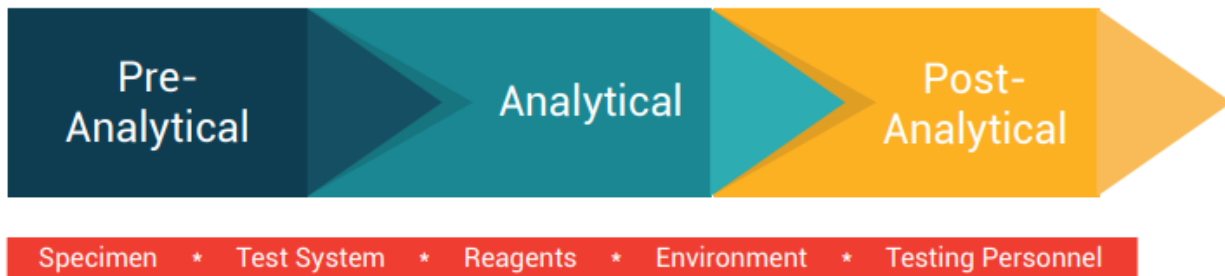

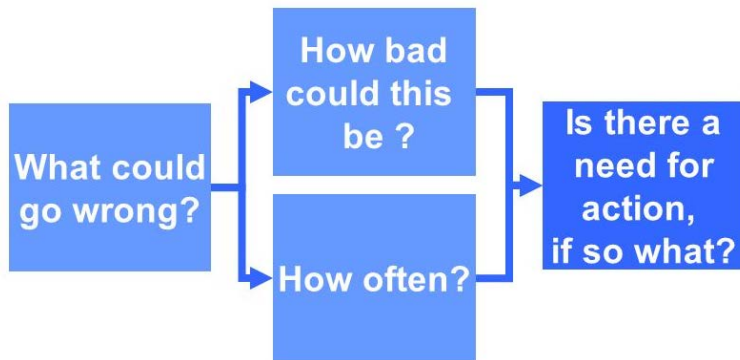

**S** – Specimen: correct collection tube, collecting collection site, adequate mixing of the specimen  
**T** – Test: follow the procedure, running CVP1 instead of CVP2 or High LQC instead of Low LQC, QC failures, errors, instrument failures  
**R** – Reagents: degrading reagents, preparing of reagents  
**E** – Environment: room temp, fridge temp, humidity  
**P** – Personnel: training and competency, user verifies that a test order in place prior to testing