ACCU-CHEK INFORM II BLOOD GLUCOSE DETERMINATION Competency Exam

This exam is to be performed upon Initial Orientation & Training and Competency; twice in the first year of practice; and annually thereafter.

PRINTED Name:	License #:	Home Unit:		
I have READ/REVIEWED the Policy prior to taking this test as indicated by my initials:				
DATE of Test:	TIME of Test:			
Test Strip Lot #:	QC Lot #:			
QC Level 1 Result:	QC Level 2 Result:			

Written Exam – Circle either T = statement is true or F = statement is false

Т	F	1. Sharing barcode IDs and/or making up MRNs is allowed in an emergency.		
Т	F	2. QC must be performed and passed every day – before a patient can be tested.		
Т	F	3. Only the night shift is responsible for performing QC testing.		
Т	F	4. Cleaning and disinfecting the meter is required between every patient.		
Т	F	5. Alternate site testing (arm or leg) has been approved for the Inform II meter.		
Т	F	6. Result documentation is the responsibility of the one who tests the patient.		
Т	F	7. The Inform II system will now automatically notify the MD of any critical result.		
Т	F	8. Unused QC solutions and test strips are to be stored in the refrigerator.		
Т	F	9. Blood samples are to be applied to the top of the test strip, not on edge of strip.		
Т	F	10. POCT glucose testing is a waived test according to the FDA.		
Т	F	11. The test strip lot # on the vial must match the test strip lot # on the meter.		
Т	F	12. QC must be performed when each new vial of test strips is first opened.		
Т	F	13. Your barcode ID contains unique identifiers imbedded in the code.		
Т	F	14. There's a "time out" imbedded in the patient identification process in the meter.		
Т	F	15. When inserting test strip in meter, place the "Accu-Chek" words facing down.		
Т	F	16. When cleaning meter and base unit, be sure to thoroughly drench meter.		
Т	F	17. If I mistrust any patient result I should repeat the patient test prior to treatment.		
Т	F	18. Plasma, serum, and cord blood are all acceptable samples for Inform II testing.		
Т	F	19. QC solutions are good for 60 days at SFGH. No policy change on this issue.		
Т	F	20. When Inform II issues arise; notify BioMed, not POCT Services.		
Passing Score is 100% -		0		
If you do not pass, reread			Passed	Failed
policy and repeat exam.				

l performed this test.	I observed this test being performed.
Printed Name:	Printed Name:
Signature:	Signature:

Fax completed test to POCT Services @ 415.206.3451 or 415.206.3045 or bring copy to 2M14.

S:\POINT OF CARE\All tests\Glucose\Initial Training & Competency Record\INFORM II Competency Test.docx