

# BinaxNOW COVID-19 Competency Exam

Initial Competency    Semi-annual Competency    Annual Competency (CY: \_\_\_\_\_)    Follow up

This exam is to be performed upon Initial Orientation & Training and Competency; twice in the first year of practice; and annually thereafter.

PRINTED Name:	License #:	Home Unit:
I have READ/REVIEWED the procedure prior to taking this test as indicated by my initials: _____		
DATE of Test:	TIME of Test:	
Pink/Purple Control QC line present: Yes or No (circle one)	Result of blind sample: Neg or Pos (circle one)	

### Written Exam – Circle either T = statement is true or F = statement is false

T	F	1. It is acceptable to use any collection swab as long as it is clean.	
T	F	2. External controls must be performed with each new shipment and new lot received and once for each untrained operator, prior to patient testing.	
T	F	3. I may still use the new test card even if the control line is not blue.	
T	F	4. Within 3 minutes of inserting the swab into the test card, I must add the extraction reagent drops.	
T	F	5. For the best results, read the test card 1 hour after closing the card.	
T	F	6. When adding the extraction reagent, I must hold the bottle vertically and add 6 drops to the Patient test card.	
T	F	7. It is acceptable to open the test card just prior to use and straight out of the refrigerator.	
T	F	8. I rotate the nasal swab shaft one time in the test card.	
T	F	9. I may return the nasal swab collection in the original paper packing.	
T	F	10. The sample can be stored in a clean, unused plastic tube at room temperature for up to one hour prior to testing.	
<b>Passing Score is 100% - If you do not pass, reread policy and repeat exam.</b>		<b>Passed</b>	<b>Failed</b>

<b><u>Employee</u></b>	<b><u>Observer</u></b>
Printed Name:	Printed Name:
Signature:	Signature: