



**MULTISTIX 4 – URINE DIPSTICK QUALITY CONTROL RECORD**

**KOVA LIQUA-TROL ABNORMAL CONTROL LEVEL I (RUN WEEKLY)**

<b>Location:</b> _____		
<b>LEVEL I QC Lot#:</b>	<b>Expiration Date:</b>	<b>Date Opened:</b>
<b>MULTISTIX Lot#:</b>	Expiration Date:	Date Opened:
<b>MULTISTIX Lot#:</b>	Expiration Date:	Date Opened:
<b>MULTISTIX Lot#:</b>	Expiration Date:	Date Opened:
<b>MULTISTIX Lot#:</b>	Expiration Date:	Date Opened:

1. Abnormal control has known pH and is positive for other tests. This will show if the dipstick is working properly.
2. **Run Control Once a Week.**
3. Verify control lot # on this form is the same lot # on control vial. Check expiration date.
4. Verify MULTISTIX lot # on this form is the same lot # on MULTISTIX vial.
5. Allow control to come to room temperature (20-30°C) for 15 minutes before testing.
6. Mix gently and then invert vial, squeeze sides, and apply control to the strip.
7. Record results on this form verifying results are within range.
8. If result(s) fall within range(s), report patient results.
9. If result(s) fall outside the range(s), repeat test. Call POC Staff if problem persists.

DATE	Leukocytes (Trace-3+)	Nitrite (Positive)	Protein (Trace-500)	Glucose (100-2000)	Performed By	Comments

**REVIEWED BY:** \_\_\_\_\_